PEAK FLOW METER

CE 0120

CLEMENT CLARKE
INTERNATIONAL

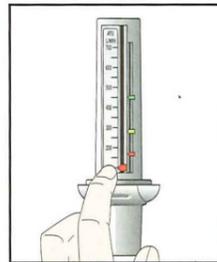
en YOUR PEAK FLOW METER
Your AirZone Peak Flow Meter will help you to monitor your asthma by measuring your peak flow. This leaflet explains how to use your Peak Flow Meter, and how to record the readings. With careful use the AirZone would be expected to last for two years.

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
1
en If required insert a mouthpiece into the meter. Ensure the pointer is set at Zero.

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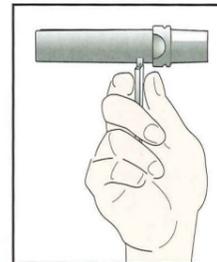
2
en Hold the Peak Flow Meter so that your fingers are clear of the scale and slot. Do not obstruct the holes at the end of the Peak Flow Meter, or the vents at the side of the mouthpiece.

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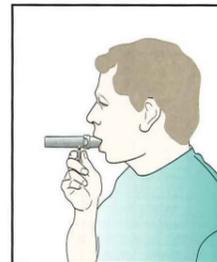
3
en Stand up if possible. Take a deep breath, place the Peak Flow Meter in the mouth and hold horizontally, closing the lips around the mouthpiece, then blow as hard and as fast as you can.

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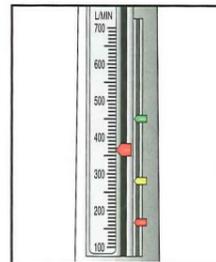
4
en Note the number on the scale indicated by the pointer.

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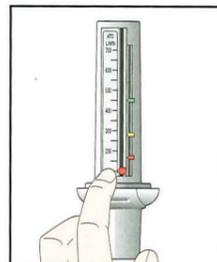
5
en Return the pointer to zero and repeat the procedure twice more to obtain three readings. Mark the highest of the three readings on the chart as shown overleaf.

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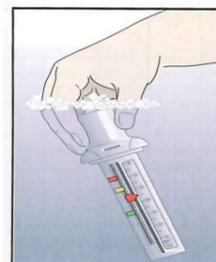en CLEANING YOUR PEAK FLOW METER

Immerse Peak Flow Meter in warm (but not hot) mild detergent solution for 2-3 minutes (maximum 5 minutes). Agitate the meter to ensure thorough cleaning.

Rinse in clean warm water and shake gently to remove any excess water. Allow to dry thoroughly before using again.

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en 1 Adult/Child AirZone Peak Flow Meter with optional Zone Markers and Instructions/ Recording Chart

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Clement Clarke International Ltd.