
Mini-Wright™

AFS Low Range Peak Flow Meter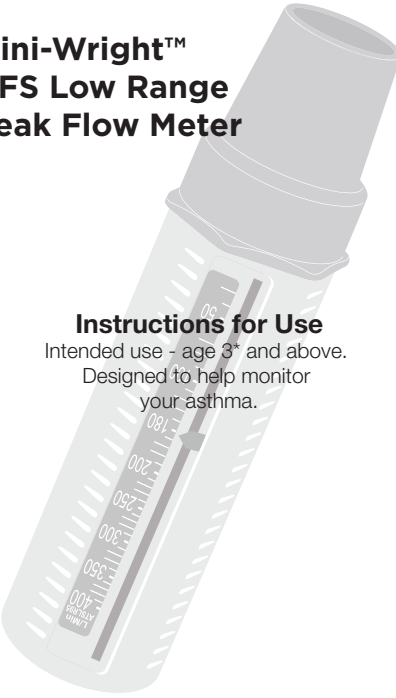

Instructions for Use

Intended use - age 3* and above.
Designed to help monitor your asthma.

Contents:

1 MINI-WRIGHT Low Range peak Flow Meter complete with detachable universal mouthpiece and instructions with peak flow diary.

en

Your Mini-Wright Low Range Peak Flow Meter will help you to monitor your asthma by measuring your peak flow. This leaflet explains how to use and care for your peak flow meter and how to record the readings. Peak flow measurements are an indication of how well air is moving through your lungs. Changes in your peak flow may require special treatment to establish what is happening in your airways.

With careful use the Mini-Wright low range peak flow meter would be expected to last for two years (2,000 uses).

- Caution**
- Do not obstruct the holes at the end of the peak flow meter.
 - Please note that any coughing or spitting into the meter will adversely affect your readings and should be avoided.
 - Fungal growths may occur if the instrument has not been cleaned for some time.
 - Store in a clean dry place.

Disposal Note:
The Mini-Wright peak flow meter can be disposed of as normal household waste.

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- How to use Your Mini-Wright Low Range Peak Flow Meter**
- Insert the mouthpiece into the meter. Ensure the pointer is at the bottom of the scale (nearest the mouthpiece).
 - Hold the peak flow meter so that your fingers are clear of the scale and slot.
 - Stand up if possible. Take a deep breath, place the peak flow meter in the mouth and hold horizontally, closing the lips around the mouthpiece, then blow as hard and as fast as you can.
 - Note the number on the scale indicated by the pointer on a piece of paper. If unusual readings are obtained and the meter does not appear broken – seek advice from your healthcare professional.
 - Return the pointer to the bottom of the scale (nearest the mouthpiece) and repeat the procedure twice more to obtain three readings. Mark the highest of the three readings on your peak flow diary.

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- Cleaning Instructions***
- Immerse the peak flow meter in warm (but not hot) mild detergent solution for 2-3 minutes (maximum 5 minutes).
 - Agitate the meter to ensure thorough cleaning.
 - Rinse in clean warm water and shake gently to remove any excess water.
 - Allow it to dry thoroughly before using again.
 - The mouthpiece is detachable and should be kept clean using a proprietary disinfectant solution.

Performance Accuracy
The Mini-Wright (low range) conforms to BS EN ISO 23747 (for the range 30 to 400 L/min) and the NAEPP guidelines.

Accuracy: Better than 10% or 10 L/min. Readings expressed as BTPS values. Resistance to flow does not exceed 0.36 kPa/l/s.

* For Multi-Patient use see Care and Use

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How to Record Your Peak Flow
It is important to keep a record of your peak flow. Three readings should be taken twice a day, first thing in the morning and at bedtime unless otherwise directed by your healthcare professional.

With a dot record the highest of the three readings on the peak flow diary provided, both in the morning and at bedtime, every day. By joining the dots (see diagram) you can see how your peak flow varies from day to day.**

Make sure to measure and note your peak flow if you suddenly become wheezy or breathless at any time of the day or night.

Care and Use
This peak flow meter does not require maintenance (other than cleaning) or calibration for the two years following its first use. At the end of two years, it should be replaced. When not in use this device should be kept in a clean, dry area away from any hazardous substances. It is important to keep your peak flow meter clean when in use. It should be cleaned at least once every six months and thoroughly if it has not been used for some time.

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The Mini-Wright low range peak flow meter is suitable for multi-patient use if Clement Clarke's validated cleaning instructions are followed. These are available on request, or via the internet at: www.clement-clarke.com

Troubleshooting
Should you take your peak flow measurement and get an unusually low reading, it could mean that your peak flow meter is broken or it could mean that the reading is accurate and your asthma is getting worse. Check to make sure that the meter is not broken by sliding the pointer along the slot, making sure that it moves freely.

If you feel comfortable that you are following the instructions correctly and that your meter is not broken, contact your healthcare professional for advice. No matter what your peak flow readings are, if you have symptoms such as chest tightness, shortness of breath, coughing or wheezing, you should contact your healthcare professional for advice.

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Clement Clarke International Ltd.
Flexicare Deutschland GmbH
Flexicare Medical Switzerland GmbH
c/o BK-Services AG Suurstoffi 22
**Clement Clarke International Ltd is